



## Technical Information for IKA IVD Tube



Consult the safety instructions of the drive unit!

### /// Abbreviations / Symbols

Symbol	Meaning
	Re-order number
	Batch code
	In-vitro diagnostic product
	Sterilized by irradiation
	Use by
	Sufficient for
	Storage humidity
	Temperature limitation at storage
	Do not reuse
	Do not use if package is damaged
	Keep the package dry
	Manufacturer
	Consult instructions for use
	CE marking

### /// Important information

#### **Notice!**

- › **BMT-20-S-IVD.50: The first packaging foil is not sterile. Please store outside of sterile areas. The second packaging foil is sterile.**
- › Sampling should be performed under sterile conditions according to the hygiene regulations of the operating theatre or laboratory.
- › On sampling the tissue should be completely covered with liquid.
- › After inserting the sample, seal the sample tube under sterile conditions according to the hygiene regulations of the operating theatre or laboratory.
- › Manual shaking not required after sample collection.
- › Before transport, store the sample tube at room temperature.
- › Transport sample to the laboratory.
- › Processing of samples using the device **IKA ULTRA-TURRAX Tube Drive** or **IKA ULTRA-TURRAX Tube Drive P control** with laminar flow.
- › The grinding time must be adapted to the form and size of the sample
- › Max. speed / Max. timer:  
**IKA ULTRA-TURRAX Tube Drive:** scale 4, 1 min.  
**IKA ULTRA-TURRAX Tube Drive P control:** 3000 rpm, 1 min.
- › Performance of microbiology and molecular diagnostic processes according to laboratory's own procedures.

### /// Intended use

- › Sterile sampling tubes for biopsies, tissue, cartilage, bone and orthopaedic samples.
- › Manufactured for single use. Suitable for microbiology and molecular diagnostics.

#### **Warning!**

##### **For professional use only!**

Do not use if there are any signs of microbial contamination, discolouration, cracks or other indications of product deterioration.

### /// Storage and shelf life

- › Sample tubes are stored at room temperature. Do not use the product beyond the expiry date indicated on the packaging.

### /// Quality inspections

- › Quality inspections are performed on each batch.

### /// Content of packaging

- › 50 sample tubes, each containing 5 ml 0.9% Sodium chloride (NaCl) solution.

### /// Order Number

› <b>BMT-20-S-IVD.50:</b>		<b>0020027186</b>
› <b>BMT-20-S-IVD1.50:</b>		<b>0020103143</b>

### /// Manufacturer



#### **IKA-Werke GmbH & Co.KG**

Janke & Kunkel-Str. 10, D-79219 Staufen

Tel. +49 7633 831-0, Fax +49 7633 831-98, sales@ika.de

**www.ika.com**

#### **IKA-Werke GmbH & Co.KG**

Janke & Kunkel-Str. 10, D-79219 Staufen

Tel. +49 7633 831-0, Fax +49 7633 831-98, sales@ika.de

**www.ika.com**